

















 <p>Dysport* (abobotulinumtoxinA)</p>	 <p>SCULPTRA*</p>	 <p>Restylane-L</p>	 <p>Restylane LYFT™</p>	 <p>Restylane SILK</p>	 <p>Restylane KYSSE™</p>	 <p>Restylane REFYNE</p>	 <p>Restylane DEFYNE</p>	 <p>Restylane CONTOUR</p>
								
<p>Dysport® (abobotulinumtoxinA) for Injection is an acetylcholine release inhibitor and a neuromuscular blocking agent indicated for the temporary improvement in the appearance of moderate to severe glabellar lines associated with procerus and corrugator muscle activity in adults <65 years of age. Dysport should be injected in a 5-point injection pattern with 10 Units/injection for a total of 50 Units.</p>	<p>Sculptra® (injectable poly-L-lactic acid) is indicated for use in people with healthy immune systems for the correction of shallow to deep nasolabial fold contour deficiencies and other facial wrinkles.</p> <p>1-2 vials/session, each session no sooner than 3 weeks. Up to 4 sessions may be needed to achieve full correction.</p>	<p>Restylane® and Restylane-L® are for mid-to-deep injection into the facial tissue for the correction of moderate to severe facial wrinkles and folds, such as nasolabial folds.</p> <p>Restylane and Restylane-L are also indicated for injection into the lips in patients over the age of 21.</p>	<p>Restylane® Lyft with Lidocaine is for deep implantation into the facial tissue for the correction of moderate to severe facial wrinkles and folds, such as nasolabial folds, for cheek augmentation, and for the correction of age-related midface contour deficiencies in patients over the age of 21. Restylane Lyft with Lidocaine is also indicated for injection into the dorsal hand to correct volume loss in patients over the age of 21.</p> <p>One 25G or one 27G 1.5" DermaSculpt or Softfil cannula. Approved for use in the midface only.</p>	<p>Restylane® Silk is for lip augmentation and for correction of perioral wrinkles in patients over the age of 21.</p> <p>One 25G or one 27G 1.5" DermaSculpt or Softfil cannula. Approved for use in lips only.</p>	<p>Restylane® Kysse is for lip augmentation and for correction of upper perioral wrinkles in patients over the age of 21.</p>	<p>Restylane® Refyne is for mid-to-deep injection into the facial tissue for the correction of moderate to severe facial wrinkles and folds, such as nasolabial folds, in patients over the age of 21.</p>	<p>Restylane® Defyne is for mid-to-deep injection into the facial tissue for the correction of moderate to severe deep facial wrinkles and folds, such as nasolabial folds in patients over the age of 21.</p> <p>Restylane Defyne is also indicated for injection into the mid-to-deep dermis (subcutaneous and/or supraperiosteal) for augmentation of the chin region to improve the chin profile in patients with mild to moderate chin retrusion in patients over the age of 21.</p>	<p>Restylane® Contour is for cheek augmentation and for the correction of midface contour deficiencies in patients over the age of 21.</p> <p>One TSK 25 or one TSK 27G 1.5" or 2" cannula. Approved for use in the midface only.</p>

Injection Areas

								
<p>Frown lines between the brows</p>	<p>Nasolabial folds, marionette lines, mental crease (chin wrinkles), pre-auricular, and sub-malar wrinkles</p>	<p>Nasolabial folds, lips, oral commissure, marionette lines</p>	<p>Nasolabial folds, oral commissure, marionette lines, cheeks/midface* contour deficiency, and subcutaneous plane in the dorsal hand</p> <p>*Can use cannula (midface only)</p>	<p>Lips* and perioral rhytids</p> <p>*Can use cannula (lips only)</p>	<p>Lips and upper perioral rhytids</p>	<p>Nasolabial folds, oral commissure, marionette lines</p>	<p>Nasolabial folds, oral commissures, marionette lines, and chin augmentation</p>	<p>Cheeks/midface* contour deficiency</p> <p>*Can use cannula</p>
								

*Please see full Important Safety Information for Dysport, including Distant Spread of Toxin Effect Boxed Warning, on the following page.